

# Physio Control CRPlus & Express AED Inspection Checklist

	January	February	March	April	May	June	July	August
<b>Date</b>								
Unit clean, no dirt or damage								
<b>AED Status indicator</b>								
Check the Status Indicator. If the Status Indicator indicates OK you are good. Any additional symbols are bad.								
<i>* The battery and/or wrench symbol indicate your AED requires immediate service</i>								
<b>AED Electrode Pads</b>								
Check the expiration date of the adult Electrode Pads. Are the pads sealed, undamaged and within expiration date? If the pads are expired, remove from the AED and replace with new pads. To order new parts call 1-877-6-AED-CPR or go to <a href="http://www.TheCPRSchool.com">www.TheCPRSchool.com</a>								
<b>Pediatric Electrode Pads</b>								
Are Pediatric Pads present? Are the pads sealed, undamaged and within expiration date? If missing, call us to order.								
<b>AED Battery - Charge-Pak</b>								
The battery (charge-pak) is replaced at the same time as the adult electrode pads. Both the adult pads and battery <b>must</b> be replaced at the same time. To order new parts call 1-877-6-AED-CPR or go to <a href="http://www.TheCPRSchool.com">www.TheCPRSchool.com</a>								
<b>Check supplies - Rescue Ready Kit</b>								
AED Rescue Ready Kit - hand towel, trauma shears, razor, CPR mask and gloves. Replace any accessories found to be damaged or that have exceeded their expiration date								
<b>AED Location</b>								
Verify that the AED is in a visible location with proper signage and that all trained responders are aware of the AED location and that it is easily accessible at all times.								
<b>Training Certification</b>								
Verify that all trained responders have current CPR AED certification training. To schedule training certification classes call 1-877-6-AED-CPR or go to <a href="http://www.TheCPRSchool.com">www.TheCPRSchool.com</a>								
<b>Inspected by:</b>								
Signature or initials of operator completing the maintenance inspection								

**For Supplies & Training - Call The CPR School at 1-877-6-AED-CPR**

E-Mail: [Keith@TheCPRSchool.com](mailto:Keith@TheCPRSchool.com)

# Philips FRx AED Inspection Checklist

Date	January	February	March	April	May	June	July	August
Unit Clean, no dirt or damage								
<b>Defibrillator Electrode Pads</b>								
Check the expiration date of the Adult Electrode Pads. Are the pads sealed, undamaged and within expiration date? If the pads are expired, remove from the AED and replace with new pads. Do you have a Pediatric Key? To order new parts call 1-877-6-AED-CPR or go to <a href="http://www.TheCPRSchool.com">www.TheCPRSchool.com</a>								
<b>Pediatric Electrode Key</b>								
Do you have the Philips FRx Infant/Child Pediatric Key? If missing, call us to order.								
<b>AED Status indicator</b>								
Check the Status Indicator. If the Status Indicator is not flashing or is flashing red a problem has been detected.								
<i>* If light is flashing Red, Beeping or not flashing at all the AED requires immediate service</i>								
<b>Check supplies - Rescue Ready Kit</b>								
AED Rescue Ready Kit - hand towel, trauma shears, razor, CPR mask and gloves. Replace any accessories found to be damaged or that have exceeded their expiration date								
<b>AED Location</b>								
Verify that the AED is in a visible location with proper signage and that all trained responders are aware of the AED location and that it is easily accessible at all times.								
<b>Training Certification</b>								
Verify that all trained responders have current CPR AED certification training. To schedule training certification classes call 1-877-6-AED-CPR or go to <a href="http://www.TheCPRSchool.com">www.TheCPRSchool.com</a>								
<b>Inspected by:</b>								
Signature or initials of operator completing the maintenance inspection								

**For Supplies & Training - Call The CPR School at 1-877-6-AED-CPR**

E-Mail: [Keith@TheCPRSchool.com](mailto:Keith@TheCPRSchool.com)

# Zoll AED Inspection Checklist

	January	February	March	April	May	June	July	August
<b>Date</b>								
Unit Clean, no dirt or damage								
<b>Defibrillator Electrode Pads</b>								
Check the expiration date of the adult Electrode Pads. Are the pads sealed, undamaged and within expiration date? If the pads are expired, remove from the AED and replace with new pads. To order new parts call 1-877-6-AED-CPR or go to <a href="http://www.TheCPRSchool.com">www.TheCPRSchool.com</a>								
<b>Pediatric Electrode Pads</b>								
Are Pediatric Pads present? Are the pads sealed, undamaged and within expiration date? If missing, call us to order.								
<b>AED Status indicator</b>								
Check the Status Indicator. Is the Status Indicator a GREEN Check Mark?								
<i>* A Red Check Mark or Beeping indicates a problem - Your AED requires immediate service</i>								
<b>Spare Batteries</b>								
Verify spare unexpired batteries? This AED requires ten (10) Duracell batteries.								
<b>Check supplies - Rescue Ready Kit</b>								
AED Rescue Ready Kit - hand towel, trauma shears, razor, CPR mask and gloves. Replace any accessories found to be damaged or that have exceeded their expiration date								
<b>AED Location</b>								
Verify that the AED is in a visible location with proper signage and that all trained responders are aware of the AED location and that it is easily accessible at all times.								
<b>Training Certification</b>								
Verify that all trained responders have current CPR AED certification training. To schedule training certification classes call 1-877-6-AED-CPR or go to <a href="http://www.TheCPRSchool.com">www.TheCPRSchool.com</a>								
<b>Inspected by:</b>								
Signature or initials of operator completing the maintenance inspection								

**For Supplies & Training** - Call The CPR School at 1-877-6-AED-CPR  
 E-Mail: [Keith@TheCPRSchool.com](mailto:Keith@TheCPRSchool.com)