

Zoll AED Inspection Checklist

AED Location:

Serial Number:

	December	January	February	March	April	May	June	July
Date								
Unit Clean, no dirt or damage								
Defibrillator Electrode Pads								
Check the expiration date of the adult and pediatric Electrode Pads. Are the pads sealed, undamaged and within expiration date? If the pads are expired, remove from the AED and replace with new pads. To order new parts call 1-877-6-AED-CPR or go to www.TheCPRSchool.com								
AED Status indicator								
Check the Status Indicator. Is the Status Indicator a GREEN Check Mark?								
<i>* A Red Check Mark or Beeping indicates a problem - Your AED requires immediate service</i>								
Spare Batteries								
Verify spare unexpired batteries? This AED requires ten (10) Duracell batteries.								
Check supplies - Rescue Ready Kit								
AED Rescue Ready Kit - hand towel, trauma shears, razor, CPR mask and gloves. Replace any accessories found to be damaged or that have exceeded their expiration date								
AED Location								
Verify that the AED is in a visible location with proper signage and that all trained responders are aware of the AED location and that it is easily accessible at all times.								
Training Certification								
Verify that all trained responders have current CPR AED certification training. To schedule training certification classes call 1-877-6-AED-CPR or go to www.TheCPRSchool.com								
Inspected by:								
Signature or initials of operator completing the maintenance inspection								

For Supplies & Training - Call The CPR School at 1-877-6-AED-CPR

E-Mail: Keith@TheCPRSchool.com